## UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

UPSTATE PET PRODUCTS INC. SAMPLE

COLLECTION	\
CARD	

UPSTATEPETPRODUCTS.COM

**SAMPLE** COLLECTION CARD

**SAMPLE** 

**CARD** 

COLLECTION

**SAMPLE** COLLECTION CARD

Species:	 	
Bird ID:	 	
Owner:		

Email:

Phone: All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the ton of the nail volume.

the top of the nail you wish to test. Have blood clotting agent ready for once test is done

enough sample may prevent us from being able to get a reliable result. Please watch video on our website so we don't have any delay on your results!

watch how to test at: upstatepetproducts.com/testing

Species:

Owner:

Bird ID:

Email: Phone:

All samples must be registered online prior to sending in results to labratory. Go to upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill)

This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the ton of the nail volume. For best results, use alcohol swab to clean all the nail clippers and the top of the nail you wish to test. Have blood cotting agent ready for once test is done

watch how to test at: upstatepetproducts.com/testing

UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

Species:\_\_\_ Bird ID:

Owner:

Email:

Phone: All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill)

This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the ton of the nail volume. enough sample may prevent us from being able to get a reliable result. Please watch video on our website so the top of the nail you wish to test. Have blood clotting agent ready for we don't have any delay on your results! once test is done

watch how to test at: upstatepetproducts.com/testing

UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

**SAMPLE** COLLECTION **CARD** 

Species:\_\_\_\_\_ Bird ID:\_\_\_\_\_

Owner:

Email:

Phone: All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill)

This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the ton of the nail volume. the top of the nail you wish to test. Have blood

enough sample may prevent us from being able to get a reliable result. Please watch video on our website so clotting agent ready for once test is done we don't have any delay on your results!

watch how to test at: upstatepetproducts.com/testing

UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

Species:\_\_\_\_

Bird ID:

Owner:

Email:

Phone: All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill)

This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the ton of the nail volume. the top of the nail you wish to test. Have blood clotting agent ready for once test is done results. Please watch video on our website so we don't have any delay on your results!

watch how to test at: upstatepetproducts.com/testing

UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

CARD

COLLECTION

**SAMPLE** 

Species:\_\_\_\_ Bird ID:

Owner:

Email:

All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill)

Phone:

This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the top of the nail you wish to test. Have blood colting agent pready for clotting agent ready for once test is done we don't have any delay on your results!

watch how to test at: upstatepetproducts.com/testing

UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

**SAMPLE** COLLECTION CARD

Species:\_\_\_\_

Bird ID:\_\_\_\_\_

Owner:

Phone: All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the top of the nail you. the top of the nail you wish to test. Have blood

Email:

enough sample may prevent us from being able to get a reliable result. Please watch video on our website so clotting agent ready for once test is done we don't have any delay on your results!

Blood should go directly inside circle indicated. Failure to produce a large watch how to test at: upstatepetproducts.com/testing UPSTATE PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

SAMPLE COLLECTION CARD

Species:\_\_\_\_ Bird ID:

Owner:

Email:

All samples must be registered online prior to sending in results to labratory. Go to upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill)

Phone:

This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the ton of the nail volume. From best results, use alcohol swab to clean all the nail clippers and the top of the nail you wish to test. Have blood colting agent ready for once test is done

watch how to test at: upstatepetproducts.com/testing

**UPSTATE** PET PRODUCTS INC.

UPSTATEPETPRODUCTS.COM

**SAMPLE** COLLECTION CARD

Species:\_\_\_\_\_

Bird ID:

Owner:

Email:

Phone: All samples must be registered online prior to sending in results to labratory. Go to

upstatepetproducts.com/dna to place orders and register your sample. Please email any questioning to (emaill) This labratory uses blood collection method from the nail of the bird. For best results, use alcohol swab to clean all the nail clippers and the top of the nail you. Blood should go directly inside circle indicated. Failure to produce a large

enough sample may prevent us from being able to get a reliable result. Please watch video on our website so the top of the nail you wish to test. Have blood clotting agent ready for we don't have any delay on your results! once test is done

watch how to test at: upstatepetproducts.com/testing